


Med-Lift and Mobility, Inc.

Lift Chair Repair Instructions for replacing a Hubbell® Hand wand

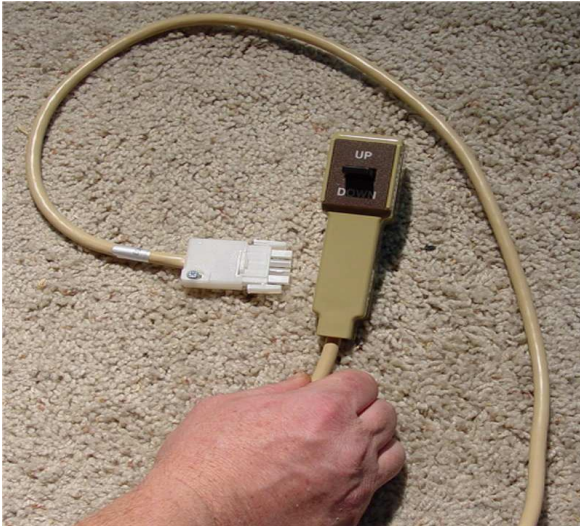

The Hubbell® hand wand is our standard 120 volt hand wand and is found on our older models and is currently used on our low through mid series lift chairs.

Tools needed for this job are:

- #2 Phillips screwdriver
- Side cutters
- Needle nose pliers
- Flat head screw driver or staple puller

Other items needed are:

- Plastic wire ties





Step 1

Unplug chair from wall outlet and turn the chair over to allow access to the underside of the chair.

Note: We recommend turning the chair over on to a clean rug or towel to prevent damaging the upholstery.

Step 2

Trace the wire coming from the hand wand to round plug coming from the base of the motor. Press the side restraint clamps, located on each side of the plug, firmly pull and unplug the hand wand from the motor.



Step 3

With the side cutters, cut the cable tie that straps the hand wand cord to the lift mechanism.





Step 4

Using your #2 Phillips screwdriver, remove the restraint screws located on the bottom of the chair base and directly below the right arm.

Note: Keep these screws and restraint clips. They will be needed for the installation of the new hand wand.

Step 5

Remove the staples, immediately around the hand wand cord, that attach the magazine pocket to the bottom of the frame.

Do this by prying staples from frame using a staple puller or flat head screwdriver.

Note: You only need to remove enough staples so that the hand wand plug that you unplugged from the motor can slide through.



Step 6

Slide hand wand through the magazine pocket and remove.

Hubbell® Hand wand replacement

Step 1

Re-insert hand wand cord through magazine pocket.




Step 2

Plug hand wand into the plug coming from motor base. Make sure the restraint clips click into place.

Step 3

Tie hand wand cable to lift mechanism using a plastic cable tie.

Note: Check and be sure wire cannot be crimped or cut by mechanism

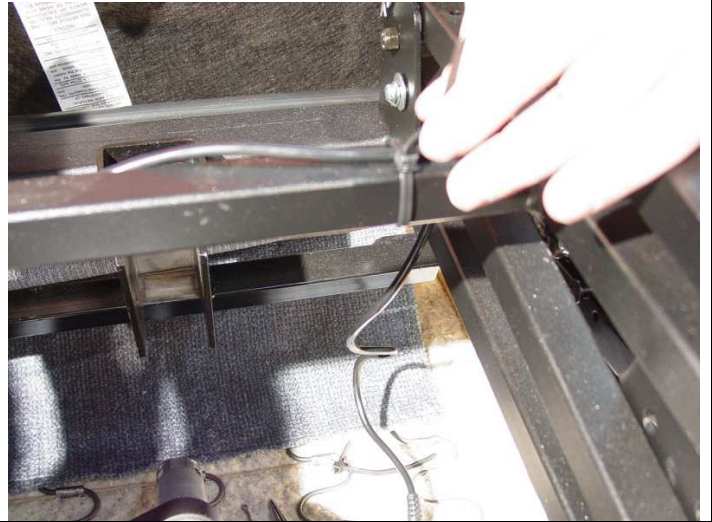

Step 4

Re-secure the hand wand cord at the bottom and below the right arm with the fasteners and Phillips head screws you removed when taking out the defective hand wand.

Make sure and use the same holes. This will make inserting the screws easier and will keep from putting more holes in the fabric under the arm.

Turn chair upright, Plug power cord in to wall plug and test to make sure chair is operating correctly



This completes the replacement of a Hubbell® hand wand.



For further assistance call Med-Lift Technical support at (800) 748-9438 or contact us at techsupport@medlift.com or at www.medlift.com.

Hubbell® and Med-Lift and Mobility® are registered trademarks