

TOUCH SENSOR SIX MOTOR MASSAGE & HEAT



R
A
F
F
E
L


S
Y
S
T
E
M
S

OPERATING INSTRUCTIONS

P7360



• www.liftchair.com

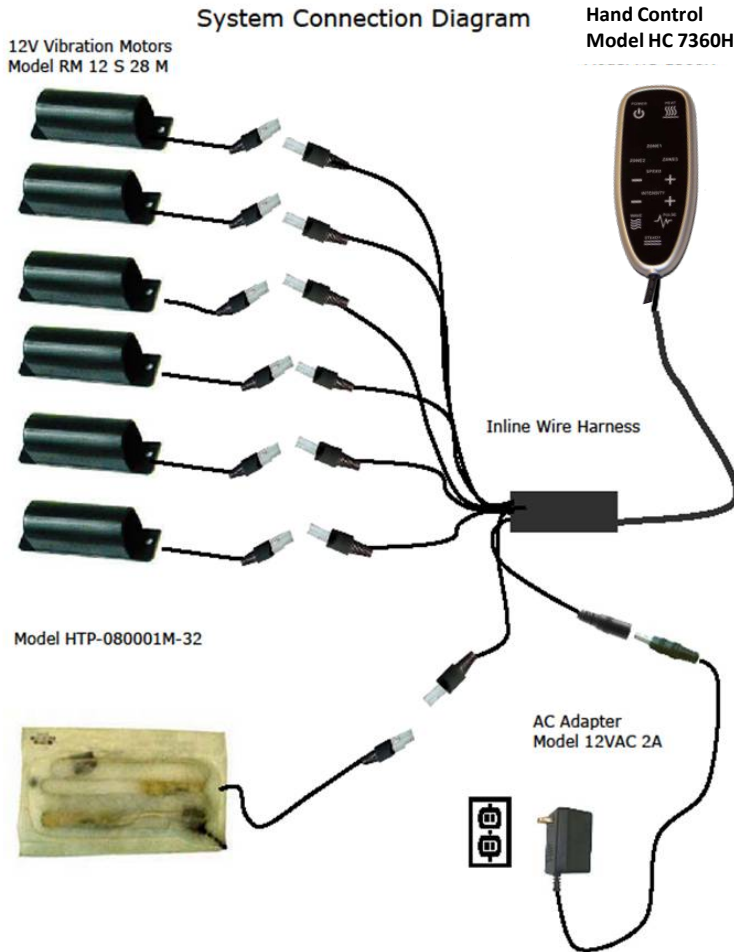

IMPORTANT SAFETY INSTRUCTIONS

WARNING: To reduce risk of injury to persons:

**READ INSTRUCTIONS BEFORE OPERATING.
INJURY MAY RESULT FROM IMPROPER USE.
NEVER REMOVE COVERS.**

- This appliance should never be left unattended when in operation.
- Unplug appliance from outlet before relocating or troubleshooting.
- Do not connect an extension cord to this unit.
- Close supervision is necessary when this appliance is used by, on, or near children, invalids, or disabled persons.
- Use this appliance only for its intended use as described in the operating instructions.
- Do not use attachments not recommended by the manufacturer.
- Keep all cords away from heated surfaces and pinch points.
- Do not use outdoors. INDOOR USE ONLY.
- Do not use on an infant or on a sleeping or unconscious person.
- Do not use on sensitive skin. Prolonged exposure may have an adverse affect on sensitive skin – Always check the skin in contact with the heated area of the appliance.
- Never operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Contact a service center for examination and/or repair.
- To disconnect, turn all controls to the off position, then remove plug from outlet.

DANGER – TO REDUCE RISK OF ELECTRIC SHOCK:

- Do not reach for an appliance that has fallen into water. Unplug immediately.
- Never use pins or other metallic fasteners with this appliance.
- Discard the appliance if the covering shows any sign of deterioration, such as checking, blistering, or cracking.
- Keep dry. Do not operate in a wet or moist environment.

P7360

Touch Sensor Six Motor Massage with Heat

Operating Instructions

Massage : Touch POWER to back light the wand.
Touch POWER , WAVE, PULSE or STEADY to
turn Massage ON. Once activated MASSAGE has a 30
minute timer. To turn off, touch Power.

Zone: Touch Zone 1, 2 or 3 to choose the location of
the massage to control.

Wave / Steady/ Pulse : Press Wave, Steady, or
Pulse keys to choose Massage type

Intensity +/- : Touch + or – keys to increase
or decrease Massage intensity

Speed +/- : Touch + or - keys to increase or
decrease the Massage speed (PULSE and
WAVE modes only)

Heat: Touch HEAT key to activate the High and again
for LOW and a third time to turn off. Once activated
there is a 30 minute timer.

Note: Use only the power supply furnished with the
unit.

